

# LIFEPAK® 20/20e DEFIBRILLATOR/MONITOR

If my LIFEPAK 20/20e device no longer prints out its 3 a.m. self test results, how do I ensure the self test is actually running?

### Explanation:

It is likely the Self Test option has been set to Off in the Auto Print menu. When Self Test is set to On, the results of the Self Test print out automatically at the end of the test.



### Steps to resolve:

To enable the Self Test results to be printed automatically after the test is complete:

1. Enter Setup mode by holding down the Event and Options buttons while pressing the On button (on the front panel of the device).
2. Enter your passcode (or try 0000) when prompted.
3. Select the Printer menu on the Setup screen.
4. Select the Auto Print menu.
5. Set the Self Test option to On.
6. Ensure the printer has paper.
7. To exit Setup Mode, power off the defibrillator.

**Note:** To use the defibrillator with any new settings, you must turn the defibrillator off and on again.

Hold down Event and Options buttons while pressing the On button

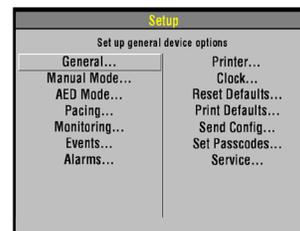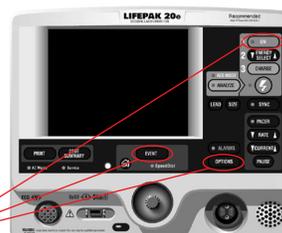

### References:

- LIFEPAK 20 defibrillator/monitor Operating Instructions, Section 8
- LIFEPAK 20e defibrillator/monitor Operating Instructions, Section 8

### Applies to:

- LIFEPAK 20 defibrillator/monitor
- LIFEPAK 20e defibrillator/monitor

For further information, please contact your Physio-Control representative at 800.442.1142 or visit our website at [www.physio-control.com](http://www.physio-control.com)