


LIFEPAK CR® Plus / LIFEPAK EXPRESS® DEFIBRILLATORS

When I try to upgrade my LIFEPAK CR Plus or LIFEPAK EXPRESS AED to the 2005 AHA guidelines with version 3.x software, why does the device quickly advise “Configuration only. No software upgrade or auto test required”?

Explanation:

This occurs because the device has already been upgraded. If you try to upgrade an AED that is already running version 3.x software, the device will display the “Configuration only” message. The screen from the LIFEPAK Defibrillator Software Solutions displays the version, which should be 3.0 or higher.

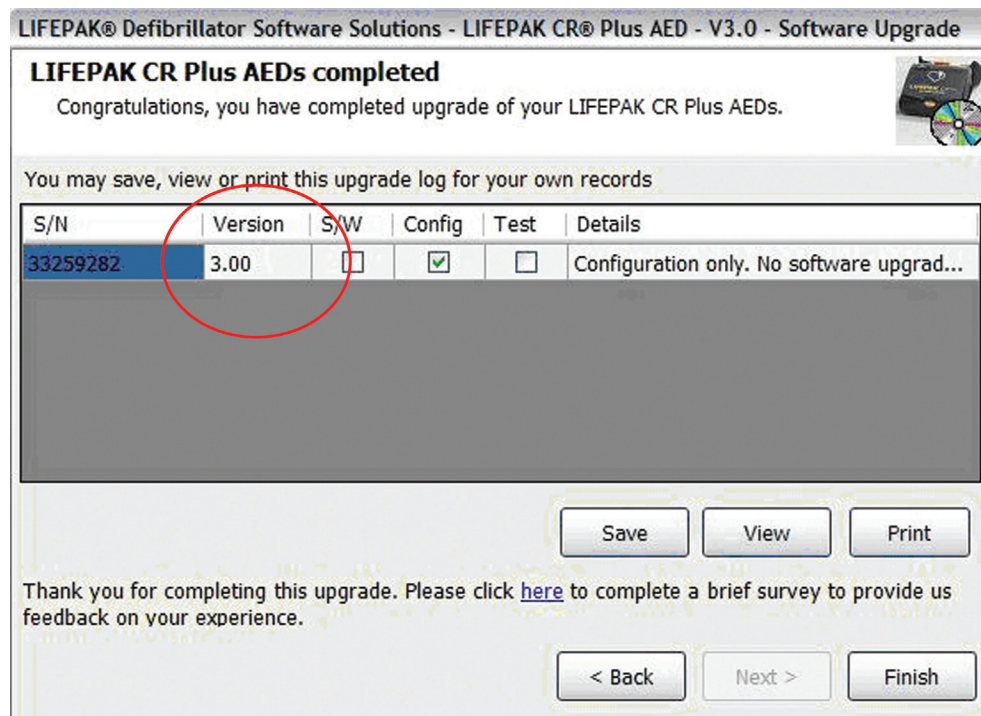

LIFEPAK CR Plus AEDs completed
 Congratulations, you have completed upgrade of your LIFEPAK CR Plus AEDs.

You may save, view or print this upgrade log for your own records

S/N	Version	S/W	Config	Test	Details
33259282	3.00	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	Configuration only. No software upgrad...

Buttons: Save, View, Print, < Back, Next >, Finish

Thank you for completing this upgrade. Please click [here](#) to complete a brief survey to provide us feedback on your experience.

Steps to resolve:

No action needed.

Applies to:

- LIFEPAK CR Plus AED
- LIFEPAK EXPRESS AED

For further information, please contact your Physio-Control representative at 800.442.1142 or visit our website at www.physio-control.com